

Nerivio[®]

by theranica


Dr.Reddy's 

Using your Nerivio[®] device

- 1 Download and install the Nerivio app.
Open the app and create an account
(refer to www.nerivio.co.uk for smartphone requirements)



- 2 Connect the device to the app

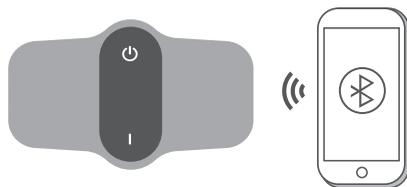

Artwork Dr.Reddy's 
Version No: v1.3

Submission

Brand: Dr. Reddy's
Country: UK
Product Name: Nerivio[®]
Strength: N/A
Form: Device
Component: Leaflet
Pack Size: 1 Nerivio[®] Device
Date Created: 08 MAR 2024
Date Modified: 10 APR 2024

Project: **NPL**

Previous DR No: N/A
Previous Material Number: N/A

Commercial

DR No: **DR001361P-3C**

DRUGS Code: N/A
Material Code: N/A
Third Party Material Code: TBC
Barcode Type: N/A
Barcode Number: N/A
Magnification: N/A
Pharmacode No: N/A

Implementation Date: N/A

Technical Information

Die Cut	Guides
	

Min Font Size: 6 pt
Printer: Theranica
Dimensions: 100 x 150 mm

Colours

Black	Cyan	Magenta	Yellow
			


Dr.Reddy's 
Good Health Can't Wait

Dr. Reddy's Laboratories (UK) Ltd,
410 Cambridge Science Park, Milton Road, Cambridge,
CB4 0PE, United Kingdom


**UNDER NO CIRCUMSTANCES
SHOULD THIS ARTWORK BE ALTERED
WITHOUT PRIOR PERMISSION FROM
DR.REDDY'S ARTWORK EU.**

Please note that any low resolution paper Canon colour copies associated with this job should be referred to for content, layout and colour separation only.


- 3** Place the device on your outer upper arm

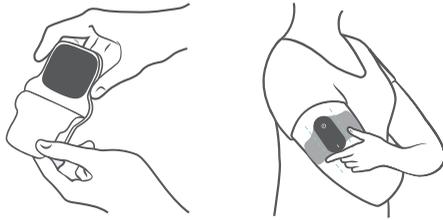

- 4** Adjust the armband to your size and wrap it around the device on your arm

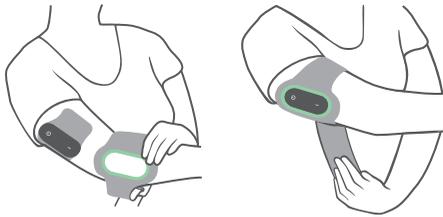

If the armband is too short, use the armband extension

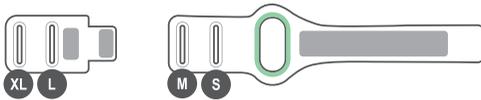

- 5** Start a treatment session within 60 minutes of migraine headache or aura onset, or every other day for preventive treatment.

Set the treatment intensity level so it feels strong yet comfortable and not painful.

